

MyCare[™] INSITE

Quick Reference



Saladax Biomedical, Inc.
116 Research Dr.
Bethlehem, PA 18015 USA
Tel: +1 610 419-6731
Fax: +1 484 547-0590


MyCareInsite.com 

OPL MCI-001-EN Rev 05

 **Saladax**
B I O M E D I C A L

Note: for a detailed video on setting up the MyCare Insite, please visit: MyCareInsite.com

MyCare Insite Packaging

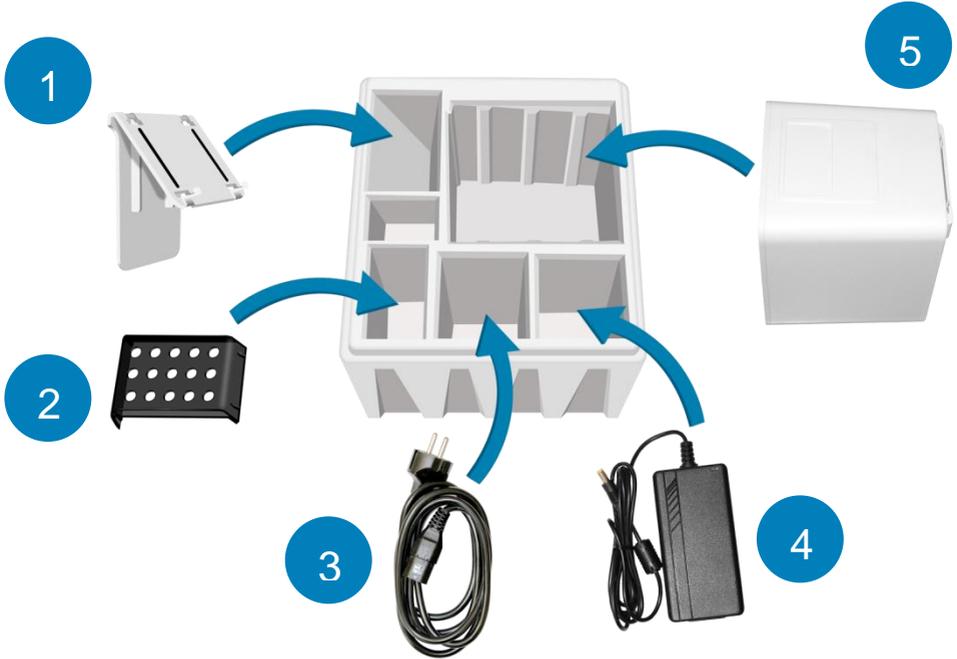

- 1. Touch Screen Stand
- 2. Test Rack
- 3. Power Cable
- 4. Power Adapter
- 5. Analyser

Transport Lock



Before first use remove the transport lock from the door of the Analyser.

Note: Keep both the transport lock and the original packaging. In case of a defect, the device must be returned in the original packaging with the transport lock installed to prevent damage in transport.

Setting Up the MyCare Insite Analyser

Plug the Power Adapter (AC adapter plug) into the Analyser and the Power Cable into the wall socket.

Analyser Back



Power Supply Port

USB Port (type B) for connecting a PC (for data transfer)

RS 232 Port for connecting a printer, a barcode reader, or a PC (for data transfer)

USB Port (type A) for connecting Touch Screen

Bluetooth ON/OFF Switch
(left: Bluetooth OFF)

**Touch Screen
Lock Port**

Indicator Lights

LEDs on the MyCare Insite Analyser

Analyser Front



This green LED indicates the Analyser status
Flashing: the Analyser is warming-up and testing cannot be started yet
(*Note:* warm-up takes approximately 10 min.)
On (green): the Analyser is warmed-up and ready for use
Off: the Analyser is not plugged in



This blue LED indicates Touch Screen connection status

On (blue): connection with Touch Screen is established
Off: the Touch Screen is not connected



This blue LED indicates Touch Screen Bluetooth connection status

On (blue): Bluetooth connection with Touch Screen is established
Flashing: Analyser Bluetooth is activated but no Touch Screen is connected via Bluetooth
Off: Bluetooth is switched off

Note: Use of the USB cable connection is recommended to keep the Touch Screen powered.

The Touch Screen

The MyCare Insite is operated with a Touch Screen.

Ensure the Analyser is plugged in. Connect the Touch Screen to the USB port of the Analyser and then turn on the Touch Screen.

Starting & Operating the Touch Screen

1. Connection Screen



The connection screen appears when the Touch Screen is connected to the Analyser.

Tap *Next* to continue.

2. Main Menu



The MyCare Insite application starts and the main menu appears.

Note: while the MyCare Insite is warming-up (indicated by the flashing green LED on the front, see page 4) the “Measurement” button is grey and inactive.

The MyCare Insite Launch Screen



The launch screen appears when the Touch Screen is not connected to the Analyser or by pressing the home button:

- Touch *Browser* to open the Android web browser
- Touch the *MyCare Insite* logo to start the MyCare Insite app
- Touch *Settings* to open the Android settings menu

Analysing a Patient Sample

1. RFID Card Placement



Take the provided RFID card out of the test kit and place it on the RFID card well on top of the Analyser.

2. Main Menu




To start a measurement, select  in the Main Menu. The MyCare Insite automatically reads the RFID card and the Test Menu appears.

3. Test Screen



The MyCare Insite displays the test name and number of tests remaining on the RFID card. Touch  to proceed. Touch  to return to the main menu.

4. Input Menu

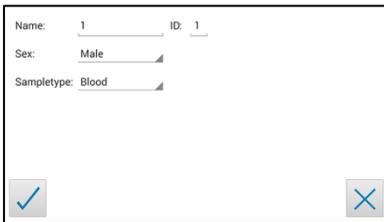

Name: 1 ID: 1
Sex: Male
Sampletype: Blood

Select the desired input field (e.g. “Name”); the Touch Screen keyboard appears. Enter the information and confirm with “Done”. Next, tap  and select an entry from the drop-down menu. Touch to confirm the input or to exit to the main menu.

5. Insert Cartridge and Close Door



The door opens automatically. Insert the cartridge (Cuvette with Reagent Cap) into the Chamber and close the door. The test starts automatically.




- Handle the Cuvette according to the instructions in the package insert
- Make sure the Reagent Cap is firmly seated into the Cuvette before inserting it into the MyCare Insite Chamber.
- Use only MyCare Insite Tests. Otherwise severe damage to the Analyser and/or incorrect results can occur.
- Do NOT attempt to open the Chamber door manually.
- Do not use liquid cleaners on the Analyser or Touch Screen. Use a lint-free, slightly damp cloth only.

Viewing and Processing Test Results

After processing a test, the results appear on the Touch Screen:

1. Result Screen

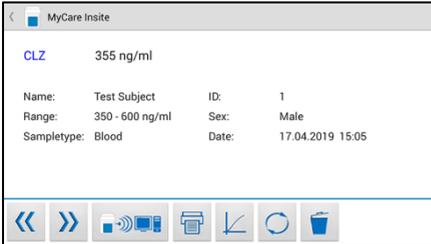

The test result is displayed on the Touch Screen. Use and to scroll.

Touch to export to a HIS/LIS or PC or send as an Email.

Touch to print.

Touch to open the Analyser data.

Touch to delete the result.

Touch to exit to the main menu.

All test results are stored in the Touch Screen.


They can be accessed as follows:

1. Main Menu



Touch in the main menu to open the Result List.

2. Result List Screen

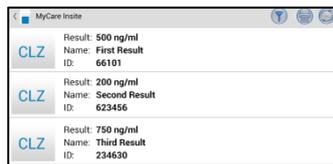


The stored results can be browsed by swiping up and down. Touch a result to open the result screen (see above); to return to the result list from a result screen press the *Return Arrow*

Touch to open the filter options.

Touch to print, mail, or export.

Touch to return to the main menu.

3. Filter Options

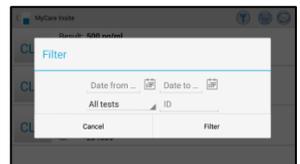

Results can be filtered by date, test, and patient ID.

Touch to set the desired start date,

to set the desired end date.

Touch to select a test in the drop-down menu (e.g. CLZ).

Touch to filter by patient ID.

How to Wake-Up the MyCare Insite or Switch It OFF



When the Touch Screen displays the main menu, the “power safe” function dims the screen. Pressing the lock switch will re-illuminate the display to its customary level of brightness. It is not necessary to switch the MyCare Insite off every day. Turning off the device completely can only be done by disconnecting it from the power supply.

Saladax Biomedical, Inc.
116 Research Dr.
Bethlehem, PA 18015 USA
Tel: +1 610 419-6731
Fax: +1 484 547-0590
MyCareInsite.com 

